

ottobock.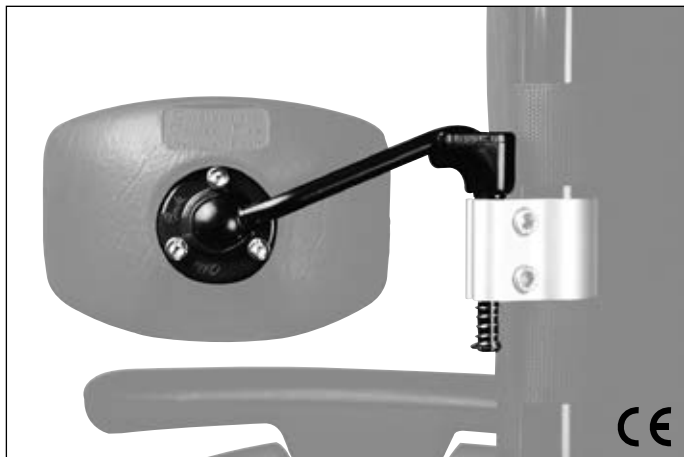


434F1, 434F2, 434F3, 434F4


DE	Gebrauchsanweisung	3
EN	Instructions for use	6
FR	Instructions d'utilisation	8
IT	Istruzioni per l'uso	11
ES	Instrucciones de uso	14
SV	Bruksanvisning	17
NO	Bruksanvisning	20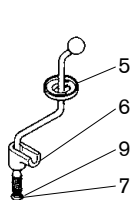


1 434F1=L
434F1=R
434F1=L-7
434F1=R-7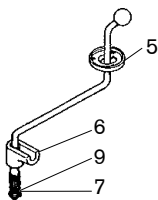

2 434F2=L
434F2=R
434F2=L-7
434F2=R-7

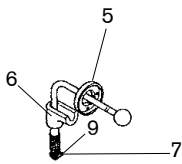3 434F3=L
434F3=R
434F3=L-7
434F3=R-7

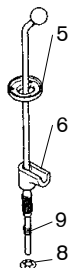
4 434F4
434F4=7


①


a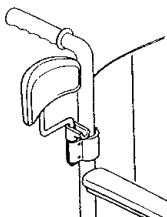

b

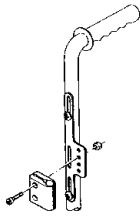c

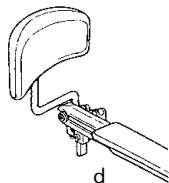d

②

INFORMATION

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INFORMATION

Last update: 2015-06-25

- Please read this document carefully before using the product.
- Instruct the user in the proper and safe use of the product.
- Please keep this document in a safe place.

INFORMATION

- New information regarding product safety and product recalls can be obtained from the Customer Care Center (CCC) at oa@ottobock.com or from the manufacturer's service department (see inside back cover or back page for addresses).
- You can request this document as a PDF file from the Customer Care Center (CCC) at oa@ottobock.com or from the manufacturer's service department (see inside back cover or back page for addresses). It is possible to increase the display size of the PDF document.

The 434F1 / 434F2 / 434F3 / 434F4 Rod Assemblies are to be used **exclusively** for mounting S.T.S. Pads to wheelchairs and seating systems.

2 Description and function

These assemblies are used for mounting S.T.S. Pads to wheelchairs or seating systems. The 434F3 Rod Assembly, dropped, can additionally be used for assembling calf legrest pads. The 434F4 Kit allows construction of a customized assembly based on the patient's requirements.

Single components (ill.1):

Item	Art. no.	Description	Quantity
1	434F1=L	Mounting Kit, left	1
	434F1=L-7	(incl. 5, 6, 7, 9)	
	434F1=R	Mounting Kit, right	
	434F1=R-7	(incl. 5, 6, 7, 9)	
2	434F2=L	Mounting Kit, left	1
	434F2=L-7	(incl. 5, 6, 7, 9)	
	434F2=R	Mounting Kit, right	
	434F2=R-7	(incl. 5, 6, 7, 9)	
3	434F3=L	Mounting Kit, left	1
	434F3=L-7	(incl. 5, 6, 7, 9)	
	434F3=R	Mounting Kit, right	
	434F3=R-7	(incl. 5, 6, 7, 9)	

Item	Art. no.	Description	Quantity
4	434F4	Ball and Attachment Rod	1
	434F4=7	(incl. 5, 6, 8, 9)	
5	434A5=5	Shaped Plate	1
6	435A4	Sleeve Lock	1
7	449A39=5/16	Retaining Ring	1
8	449A29=5/16	Lock Washer	2
9	513D52=57	Compression spring	1

3 Assembly

For custom fabrication of the 434F4 you need a bending tool (711S6). Clamp the bending tool in a vise. Determine the angle needed for the fixation rod and mark the points where bends are needed. This mark should be placed on the inside curve of the bend. Place the rod in the bending tool, and centre the adjustment screw directly over the bend mark. Wear eye protection. Tighten the screw until the desired angle is achieved. If two or more bendings are required, it is best to sequence the bends starting at the end of the hardware which is intended to retain the assembly part. Make sure to allow at least 3.5", i.e. 89 mm, after the bend if you must cut the rod. Individual bending thus allows customized rod assemblies.

Mounting the rod assemblies:

- 1) First mount the pad/calf legrest pad to the rod assembly by means of the slotted washer and three socket head cap screws.
- 2) Then mount the S.T.S. Clamps (434H1) which must have the suitable dimension for the wheelchair's tube diameter (fig. 2, b).
For wheelchairs with rigid back (wood plate or plastic shell) or for wheelchairs where the back upholstery sheathes the back frame tubes you need a Front Mount (434H4=7042) (fig. 2, c).
With the MOSS System the clamps can be mounted to the frame channel using socket head cap screws and nuts (fig. 2, a).
The rod assembly can also be mounted to the armrest fixture (fig. 2, d).
- 3) Mount the set in the clamps. The compression spring (Fig. 1, no. 9) and the lock washer (Fig. 1, no. 8) must be mounted first with the 434F4 assembly. The lock washer should compress the compression spring sufficiently so that the rod is held in the sleeve lock. However, additional compression of the compression spring must be possible so that the rod can be lifted out of the sleeve lock to swing it away. (The compression spring and lock washer are already mounted on the other rod assemblies).
- 4) The knurled end of the sleeve lock inserts into the clamp.

4 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

4.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

4.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock Mobility Solutions GmbH