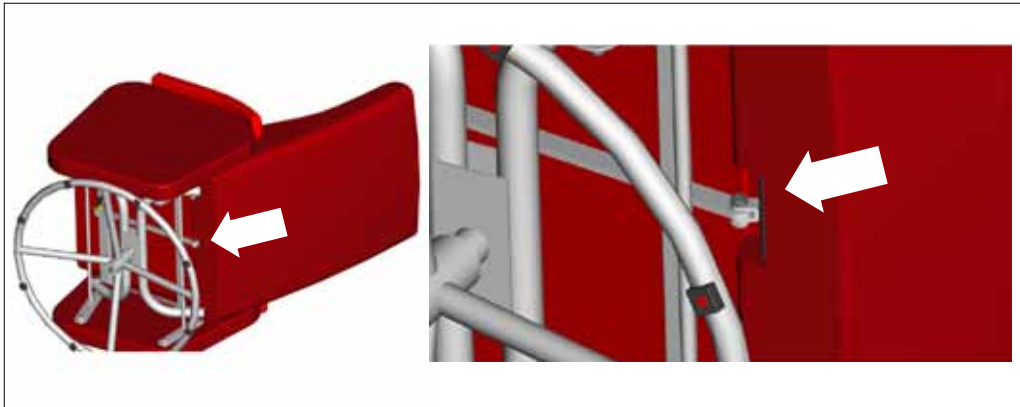
ThevoChair

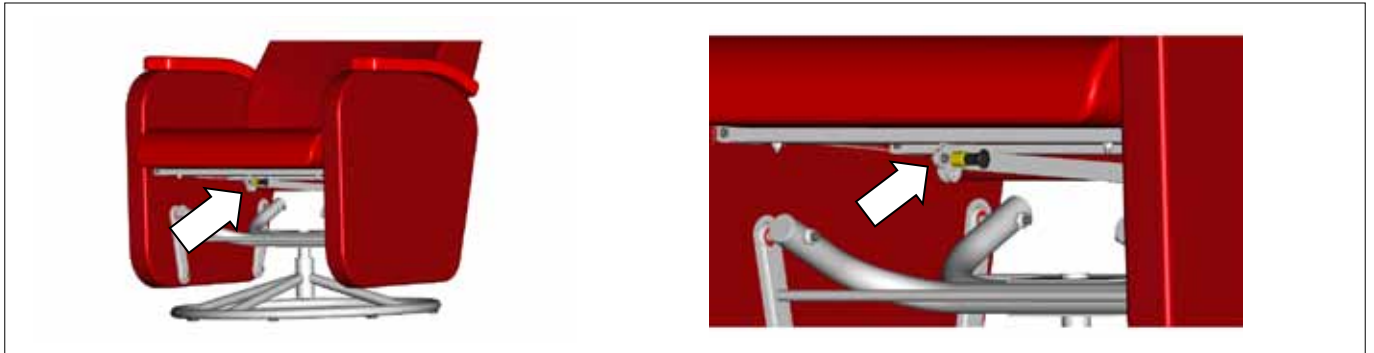
DE	Bedienerhandbuch – ThevoChair.....	3
GB	User manual – ThevoChair.....	7
FR	Manuel d’utilisation – ThevoChair.....	11
ES	Manual de usuario – ThevoChair.....	15
IT	Manuale d’istruzioni – ThevoChair.....	19
NL	Gebruiksaanwijzing – ThevoChair.....	23
NO	Brukerhåndbok – ThevoChair.....	27
SE	Bruksanvisning – ThevoChair.....	31
DK	Brugsanvisning – ThevoChair.....	35
FI	Käyttäjän käsikirja – ThevoChair.....	39

1

2

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Dear User,

You have just bought the ThevoChair, a product of the highest quality. The ThevoChair uses the most modern materials, guaranteeing a long life.

Before you first use the ThevoChair, it is essential that you should carefully read the User Manual, which contains much helpful information and many tips.

If you have any questions or suggestions about the ThevoChair, please ring us up.

You can find further information under **www.thevo.info**

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Contents

	Page
General notes on safety and warnings	8
Scope of delivery	8
Technical data	8
First operation	9
Basic settings.....	9
Accessories.....	9
Disinfection methods and recommendations for cleaning.....	10
Risks during use and contraindications.....	10
Conditions of guarantee.....	10

General notes on safety and warnings

Please do not expose your ThevoChair to excessive stress. Refer to the maximal load-bearing capacity in the technical data.

Caution: Do not insert your fingers into the rocking device.

The ThevoChair should only be placed or used on an even surface. Do not use the ThevoChair in places where it will be exposed to water or high temperatures. Do not place heavy objects on the ThevoChair.

Do not sit or stand on the stool (accessories), as the supporting surface is sloping and there is a risk of slipping off. Keep these instructions carefully! Only use original spare parts from the manufacturer.

Do not undo or remove any screws, unless this is given in the instructions.

Scope of Delivery

- ThevoChair
- User Manual
- Accessories [as ordered]

Technical data

ThevoChair	
Height of back	72 cm / 28.4"
Height	110 cm / 43.3"
Breadth (width)	73 cm / 28.7"
Depth of seat	47 cm / 18.5"
Width of seat	55 cm / 18.5"
Back angle	112° / 115°
Seat height	48 cm / 18.9"
Max. load-bearing capacity	140 kg / 308.6 lbs
Foot Stool (accessory)	
Height (max)	41 cm / 16.1"
Breadth	44 cm / 17.3"
Depth	44 cm / 17.3"

First operation

Necessary tools: 1 spanner SW 13 (included in delivery), 1 Allen screw SW 5 (included in delivery)

After you have unpacked the ThevoChair, you only have to connect the back rest with the anchor bar to be able to use the ThevoChair. The necessary steps are as follows:

1. Tip the ThevoChair carefully on the side and lay it on a clean base.
2. Set up the back rest.
3. Connect the anchor bar and the bracket flap to the back rest with the enclosed screw and fasten with the self-locking nut. Important: The thread of the screw must slightly protrude above the nut. (Fig. 1)
4. Carefully set up the ThevoChair again.

Your ThevoChair is now ready for use.

Basic Settings

Adjusting the back rest

The angle of the back rest can be set at two different positions. The factory setting is in the steeper position. The arm rest can be tipped backwards by unlocking the anchor bar by pulling the tensile latch bolt in the middle of the front edge of the seat. Let the latch bolt go and lock the back rest in the new position. (Fig. 2)

Accessories

We offer the following accessories for the ThevoChair:

- Foot Stool
- protective incontinence covers for ThevoChair + Foot Stool
- Cover for the foot of the chair

Attachment of the cover for the foot of the chair (accessories)

You can mount the cover for the foot of the chair as follows:

1. Tip the ThevoChair carefully on the side and lay it on a clean base.
2. Open the 4 zips of the cover.
3. Pull the cover over the foot of the chair and fasten all zips.
4. Make sure that the cover is sitting evenly.
5. Carefully set up the ThevoChair again - ready

Attachment of the protective incontinence covers (accessories)

The set contains five separate covers which can be used for the backrest, the seating surface and the side panel of the ThevoChair as well as a cover for the surface of the footstool. We recommend using the protective covers for extended use especially, e.g. in care communities. Please find the separate instruction enclosed at the accessories.

You can order accessories for the ThevoChair from Thomashilfen. We would be glad to advice you.

Disinfection methods and recommendations for cleaning

You have just purchased a product made of high quality materials. This requires proper care.

We wish to point out that even high quality materials can bleach if exposed to intense sunlight for long periods.

For extended use, e.g. in care communities, we recommend using our protective covers. You cannot wash the covers separately. Do not attempt to remove the covers! You can clean the ThevoChair by wiping with disinfectant.

Risks during use and contraindications

ThevoChair was developed to improve the perception and symptoms of patients with dementia.

No special instructions are needed for using the ThevoChair. No problems of any sort have cropped up during the practical use of this product or during extensive testing.

There are no risks in the use of this product if it is used properly.

Conditions of Guarantee

The company Thomashilfen vouches for the quality of the product delivered. The selection of the material and the processing are based on the best possible production standards. Your ThevoChair has been submitted to thorough final testing.

Thomashilfen gives you a guarantee of 10 years on wearing out the seat and the rocking function of ThevoChair. A prerequisite is, however, that you send in the defect product including the proof of purchase before the guarantee expires. Consider this ThevoChair as being as good as new for the first 3 years. From then on we must, in case of any guarantee undertaken on our part, charge up a value loss of 10 % for each year of use against the purchase price. Please understand that for the upholstered fabric and other product components, which are not defined as the seat and the rocking function, the legal warranty period applies.

The guarantee covers all claims arising from defects in processing and materials which impair the function. Damage is excluded if it arises from improper treatment or natural wear. Installation and transportation costs arising from an undertaking of guarantee on our part following expiration of the first warranty year must, unfortunately, be charged to the customer.

We request you carefully to store all records connected with the ThevoChair.

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Thomashilfen gives you a guarantee of 10 years on wearing out the seat and the rocking function of ThevoChair. A prerequisite is, however, that you send in the defect product including the proof of purchase before the guarantee expires.

Consider this ThevoChair as being as good as new for the first 3 years. From then on we must, in case of any guarantee undertaken on our part, charge up a value loss of 10 % for each year of use against the purchase price. Please understand that for the upholstered fabric and other product components, which are not defined as the seat and the rocking function, the legal warranty period applies.

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The set contains five separate covers which can be used for the backrest, the seating surface and the side panel of the ThevoChair as well as a cover for the surface of the footstool. We recommend using the protective covers for extended use especially, e.g. in care communities. Please find the separate instruction enclosed at the accessories. You can order accessories for the ThevoChair from Thomashilfen. We would be glad to advice you.

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